

Noritake Value Shades are additional shades that expand the existing EX-3 and CZR selection of Body and Opacious Body porcelain powders to include the VITA® 3D-Master® shades. With the Noritake Value Shades you will discover a simple, quick, and precise solution to completing VITA® 3D-Master® shade restorations.

Color Combination Table Converting VITA® 3D-Master® Shades to Noritake Value Shades

Super Porcelain EX-3		DENTAL PORCELAIN for Metal Framework Restorations					
VITA® 3D-Master® Shade	Noritake Value Shade	Opaque (Universal Paste Opaque / Paste Opaque)	Margin	Body	Opacious Body	Enamel	Translucent / Luster
0M1	—	UPNW ₀ / PONW ₀	MNW ₀	NW ₀ B	—	Silky E ₂	T ₁ / LT ₁
0M2	—	UPNW ₀ / PONW ₀	MNW _{0.5}	NW _{0.5} B	—	Silky E ₂	T ₁ / LT ₁
0M3	—	UPNW _{0.5} / PONW _{0.5}	MNW _{0.5}	NW _{0.5} B	—	Silky E ₂	T ₁ / LT ₁
1M1	NV1110	UPNW _{0.5} / PONW _{0.5}	MA ₁	1110B	OB1110	E ₁	T ₁ / LT ₁
1M2	NV1120	UPnB ₁ / POnB ₁	MA ₁	1120B	OB1120	E ₁	T ₁ / LT ₁
2L1.5	NV2015	UPnB ₂ / POnB ₂	MB ₂	2015B	OB2015	E ₁	T ₁ / LT ₁
2L2.5	NV2025	UPnB ₂ / POnB ₂	MB ₂ +MD ₄ *1	2025B	OB2025	E ₁	T ₁ / LT ₁
2M1	NV2110	UPnB ₁ / POnB ₁	MC ₁	2110B	OB2110	E ₁	T ₁ / LT ₁
2M2	NV2120	UPnB ₂ / POnB ₂	MB ₂	2120B	OB2120	E ₁	T ₁ / LT ₁
2M3	NV2130	UPnB ₂ / POnB ₂	MB ₃	2130B	OB2130	E ₁	T ₁ / LT ₁
2R1.5	NV2215	UPnA ₁ / POnA ₁	MA ₂	2215B	OB2215	E ₁	T ₁ / LT ₁
2R2.5	NV2225	UPnA ₂ / POnA ₂	MA ₂	2225B	OB2225	E ₁	T ₁ / LT ₁
3L1.5	NV3015	UPnC ₂ / POnC ₂	MC ₂	3015B	OB3015	E ₂	T ₁ / LT ₁
3L2.5	NV3025	UPnB ₃ / POnB ₃	MB ₃ +MD ₄ *1	3025B	OB3025	E ₂	T ₁ / LT ₁
3M1	NV3110	UPnC ₁ / POnC ₁	MC ₂	3110B	OB3110	E ₂	T ₁ / LT ₁
3M2	NV3120	UPnB ₃ / POnB ₃	MB ₃	3120B	OB3120	E ₂	T ₁ / LT ₁
3M3	NV3130	UPnB ₃ / POnB ₃	MB ₄	3130B	OB3130	E ₂	T ₁ / LT ₁
3R1.5	NV3215	UPnD ₃ / POnD ₃	MA ₃ +MC ₂ *1	3215B	OB3215	E ₂	T ₁ / LT ₁
3R2.5	NV3225	UPnA ₃ / POnA ₃	MA ₃ +MC ₂ *1	3225B	OB3225	E ₂	T ₁ / LT ₁
4L1.5	NV4015	UPnC ₂ / POnC ₂	MA ₄ +MC ₂ *1	4015B	OB4015	E ₂	T ₁ / LT ₁
4L2.5	NV4025	UPnB ₃ / POnB ₃	MA ₄ +MB ₃ *1	4025B	OB4025	E ₂	T ₁ / LT ₁
4M1	NV4110	UPnC ₂ / POnC ₂	MC ₂	4110B	OB4110	E ₂	T ₁ / LT ₁
4M2	NV4120	UPnB ₃ / POnB ₃	MA ₄	4120B	OB4120	E ₂	T ₁ / LT ₁
4M3	NV4130	UPnB ₄ / POnB ₄	MA ₄	4130B	OB4130	E ₂	T ₁ / LT ₁
4R1.5	NV4215	UPnA ₂ / POnA ₂	MD ₃ +MC ₂ *1	4215B	OB4215	E ₂	T ₁ / LT ₁
4R2.5	NV4225	UPnB ₄ / POnB ₄	MA ₄	4225B	OB4225	E ₂	T ₁ / LT ₁
5M1	NV5110	UPnA ₄ / POnA ₄	MA ₄	5110B	OB5110	E ₂	T ₁ / LT ₁
5M2	NV5120	UPnA ₄ / POnA ₄	MA ₄	5120B	OB5120	E ₂	T ₁ / LT ₁
5M3	NV5130	UPnA ₄ / POnA ₄	MA ₄	5130B	OB5130	E ₂	T ₁ / LT ₁

*1 / Mixing ratio is 1 : 1

CERABIEN		DENTAL PORCELAIN for Zirconia Framework Restorations										
VITA® 3D-Master® Shade	Noritake Value Shade	Noritake Zirconia Shades	The Ideal Frame Shades	Shade Base Stain	Shade Base *1	Margin	Body	Opacious Body	Enamel	Translucent / Luster		
0M1	—	ZP/KD10	Non Colored (White)	SSNW ₀	SBNW ₀	MNW ₀	NW ₀ B	—	Silky E ₂	T ₁ / LT ₁		
0M2	—			SSNW ₀	SBNW ₀	MNW _{0.5}	NW _{0.5} B	—	Silky E ₂	T ₁ / LT ₁		
0M3	—			SSNW _{0.5}	SBNW ₀ +SBNW _{0.5} *2	MNW _{0.5}	NW _{0.5} B	—	Silky E ₂	T ₁ / LT ₁		
1M1	NV1110	ZP/KD10	Non Colored (White)	SSC ₁	SBC ₁	MA ₁	1110B	OB1110	E ₁	T ₁ / LT ₁		
1M2	NV1120			SSA ₁	SBA ₁	MA ₁	1120B	OB1120	E ₁	T ₁ / LT ₁		
2L1.5	NV2015			ZP/KD12	A ₁ , B ₁ , B ₂	SSB ₂	SBB ₂	MB ₂	2015B	OB2015	E ₁	T ₁ / LT ₁
2L2.5	NV2025	SSB ₂	SBB ₂			MB ₂ +MD ₄ *2	2025B	OB2025	E ₁	T ₁ / LT ₁		
2M1	NV2110	SSC ₁	SBC ₁			MC ₁	2110B	OB2110	E ₁	T ₁ / LT ₁		
2M2	NV2120	SSB ₂	SBB ₂			MB ₂	2120B	OB2120	E ₁	T ₁ / LT ₁		
2M3	NV2130	SSB ₃	SBB ₃			MB ₃	2130B	OB2130	E ₁	T ₁ / LT ₁		
2R1.5	NV2215	SSA ₁	SBA ₁			MA ₂	2215B	OB2215	E ₁	T ₁ / LT ₁		
2R2.5	NV2225	SSA ₂	SBA ₂			MA ₂	2225B	OB2225	E ₁	T ₁ / LT ₁		
3L1.5	NV3015	ZP/KD13	A ₂ , A ₃			SSC ₂	SBB ₃ +SBC ₂ *2	MC ₂	3015B	OB3015	E ₂	T ₁ / LT ₁
3L2.5	NV3025					SSB ₃	SBB ₃	MB ₃ +MD ₄ *2	3025B	OB3025	E ₂	T ₁ / LT ₁
3M1	NV3110					SSC ₁	SBC ₁	MC ₂	3110B	OB3110	E ₂	T ₁ / LT ₁
3M2	NV3120			SSB ₂	SBB ₂	MB ₃	3120B	OB3120	E ₂	T ₁ / LT ₁		
3M3	NV3130			SSB ₃	SBB ₃	MB ₄	3130B	OB3130	E ₂	T ₁ / LT ₁		
3R1.5	NV3215			SSA ₂	SBA ₁	MA ₃ +MC ₂ *2	3215B	OB3215	E ₂	T ₁ / LT ₁		
3R2.5	NV3225			SSA ₃	SBA ₃	MA ₃ +MC ₂ *2	3225B	OB3225	E ₂	T ₁ / LT ₁		
4L1.5	NV4015			ZP/KD15	C ₁ , D ₂	SSC ₂	SBC ₂	MA ₄ +MC ₂ *2	4015B	OB4015	E ₂	T ₁ / LT ₁
4L2.5	NV4025					SSB ₃	SBB ₃	MA ₄ +MB ₃ *2	4025B	OB4025	E ₂	T ₁ / LT ₁
4M1	NV4110					SSC ₂	SBC ₂	MC ₂	4110B	OB4110	E ₂	T ₁ / LT ₁
4M2	NV4120	SSA ₄	SBB ₃			MA ₄	4120B	OB4120	E ₂	T ₁ / LT ₁		
4M3	NV4130	SSA ₄	SBB ₄			MA ₄	4130B	OB4130	E ₂	T ₁ / LT ₁		
4R1.5	NV4215	SSA ₂	SBA ₂			MD ₃ +MC ₂ *2	4215B	OB4215	E ₂	T ₁ / LT ₁		
4R2.5	NV4225	SSA ₄	SBB ₄			MA ₄	4225B	OB4225	E ₂	T ₁ / LT ₁		
5M1	NV5110	ZP/KD15	C ₁ , D ₂			SSC ₂	SBC ₂	MA ₄	5110B	OB5110	E ₂	T ₁ / LT ₁
5M2	NV5120					SSA ₄	SBA ₄	MA ₄	5120B	OB5120	E ₂	T ₁ / LT ₁
5M3	NV5130					SSA ₄	SBA ₄	MA ₄	5130B	OB5130	E ₂	T ₁ / LT ₁

*1 / In case of masking Zirconia frameworks *2 / Mixing ratio is 1 : 1