

Foil heating times for Thermoforming unit of SCHEU-Dental

Description	Order no.	Contents	Order no.	Contents	Heating time	Code
Erkolen, 0.5 mm, transparent	511505	20 pieces	514505	100 pieces	30 sec	122
Erkolen, 0.6 mm, transparent	511506	20 pieces	514506	100 pieces	35 sec	132
Erkolen, 0.7 mm, transparent	511507	20 pieces	514507	100 pieces	40 sec	142
Erkolen, 1.0 mm, transparent	511510	20 pieces	514510	100 pieces	45 sec	153
Erkolen, 2.0 mm, transparent	511520	10 pieces	514520	50 pieces	100 sec	253
Erkodur, 0.5 mm, clear	521505	20 pieces	524505	100 pieces	25 sec	111
Erkodur, 0.6 mm, clear**	521506	20 pieces	524506	100 pieces	25 sec	112
Erkodur, 0.8 mm, clear	521508	20 pieces	524508	100 pieces	30 sec	122
Erkodur, 1.0 mm, clear**	521510	20 pieces	524510	100 pieces	30 sec	122
Erkodur, 1.5 mm, clear**	521515	10 pieces	524515	50 pieces	45 sec	153
Erkodur, 2.0 mm, clear**	521520	10 pieces	524520	50 pieces	65 sec	194
Erkodur, 2.5 mm, clear	521525	10 pieces	524525	50 pieces	75 sec	217
Erkodur, 3.0 mm, clear	521530	10 pieces	524530	50 pieces	80 sec	227
Erkodur, 4.0 mm, clear	521540	10 pieces	524540	50 pieces	110 sec	257
Erkodur-S, 0.8 mm, clear	522908	20 pieces			25 sec	111
UZF, 0.10 mm, red	532501	100 pieces			10 sec	100
Erkoplast PLA-R, 1.5 mm, rosa	565615	10 pieces	565715	50 pieces	30 sec	124
Erkoplast PLA-W, 3.0 mm, white	567230	10 pieces	567430	50 pieces	85 sec	236
Erkoplast PLA-W, 4.0 mm, white	567240	10 pieces	567440	50 pieces	105 sec	256
Erkoplast PLA-T, 3.0 mm, transparent	569230	10 pieces	569430	50 pieces	85 sec	236
Erkoplast PLA-T, 4.0 mm, transparent	569240	10 pieces	569430	50 pieces	105 sec	256
Erkoflex-bleach, 1.0 mm	581510	20 pieces	584510	100 pieces	50 sec	162
Erkoflex-95, 1.5 mm, transparent	585915	10 pieces	585415	50 pieces	60 sec	183
Erkoflex-95, 2.5 mm, transparent	585925	10 pieces	585425	50 pieces	95 sec	244
Erkoflex-95, 4.0 mm, transparent	585940	10 pieces	585440	50 pieces	140 sec	287
Erkoflex, 1.0 mm, transparent	581910	20 pieces	584910	100 pieces	30 sec	122
Erkoflex, 1.5 mm, transparent	581915	10 pieces	584915	50 pieces	40 sec	142
Erkoflex, 2.0 mm, transparent*	581920	10 pieces	584920	50 pieces	50 sec	163
Erkoflex, 3.0 mm, transparent	581930	10 pieces	584930	50 pieces	70 sec	205
Erkoflex, 4.0 mm, transparent*	581940	10 pieces	584940	50 pieces	105 sec	256
Erkoflex, 5.0 mm, transparent	581950	10 pieces	584950	50 pieces	115 sec	277
Erkolign, 1.0 mm, transparent	541510	20 pieces	544510	100 pieces	55 sec	175
Erkolign, 2.0 mm, transparent	541520	10 pieces	544520	50 pieces	120 sec	266
Erkoloc-pro, 1.0 mm, transparent	595510	20 pieces			45 sec	152
Erkoloc-pro, 1.3 mm, transparent	595513	20 pieces			50 sec	162
Erkoloc-pro, 2.0 mm, transparent + <i>blu</i>	595520	10 pieces	596620	10 pieces	60 sec	185
Erkoloc-pro, 3.0 mm, transparent + <i>blu</i>	595530	10 pieces	596630	10 pieces	90 sec	236
Erkoloc-pro, 4.0 mm, transparent + <i>blu</i>	595540	10 pieces	596640	10 pieces	110 sec	257
Erkoloc-pro, 5.0 mm, transparent + <i>blu</i>	595550	10 pieces	596650	10 pieces	150 sec	287
Erkoloc-pro, 2.0 mm, green + <i>pink</i>	598220	10 pieces	597520	10 pieces	60 sec	185
Erkoloc-pro, 3.0 mm, green + <i>pink</i>	598230	10 pieces	597530	10 pieces	90 sec	236
Erkoloc-pro, 4.0 mm, green + <i>pink</i>	598240	10 pieces	597540	10 pieces	110 sec	257
Erkoloc-pro, 5.0 mm, green + <i>pink</i>	598250	10 pieces	597550	10 pieces	150 sec	287

* For coloured Erkoflex the heating times may vary depending on the machine type.

Heat coloured Erkoflex as long as the foil sags app. 2 cm, then thermoform!

** For ERKODUR-A1/A2/A3+0M1 (tooth colour) and ERKODUR freeze (turquoise) the same heating times are applicable as for ERKODUR clear.

Complete Delivery program Ø 125 mm - see Material card (www.erkodent.com/dental/download/materialkarten/Material_EN.pdf)